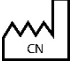
NATRACURE™ ADVANCED COOLING GEL CAP 1 Piece 11.8" x 7.5" x 2.0"	REF 762-00NC1
NATRACURE™ AVANCERET KØLENDE GELHÆTTE 1 Stk. 11,8" x 7,5" x 2,0"	BONNET DE GEL DE REFROIDISSEMENT NATRACURE™ AVANCÉ 1 unité 11,8 po (30 cm) x 7,5 po (19 cm) x 2,0 po (5 cm)
NATRACURE™ AUSGEREIFTE KÜHLGELKAPPE 1 Stück 11,8" x 7,5" x 2,0"	CAPPELLO IN GEL A RAFFREDDAMENTO AVANZATO NATRACURE™ 1 unità 11,8" x 7,5" x 2,0"
NATRACURE™ AVANSERT KJØLEGEHETTE 1 del 30 x 19 x 5 cm	NATRACURE™ CAPUZ DE GEL DE REFRIGERAÇÃO AVANÇADA 1 Peça 11.8" x 7.5" x 2.0"
NATRACURE™ ADVANCED COOLING GEL CAP 1 Piece 11.8" x 7.5" x 2.0"	NATRACURE™ HUVUDBAND MED AVANCERAD KYLANDE GEL 1 st. 11,8" x 7,5" x 2,0"

 QualRep Services BV Utrechtseweg 310 – B42, 6812 AR, Arnhem – the Netherlands Phone: +31 20 788 2630 globalreg@qservegroup.com	 PolyGel, LLC 1 Danforth Drive Easton, PA 18045 USA	
QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Holland Telefon: +31 20788 2630	QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Pays-Bas Téléphone: +31 20788 2630	 YYYY-MM-DD
QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Paesi Bassi Telefono: +31 20788 2630	QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Niederlande Telefon: +31 20 788 2630	 XXXXX
QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Holanda Telefone: +31 20 788 2630	QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Nederland Telefon: +31 20788 2630	    
QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Nederlanderna Telefon: +31 20788 2630	Servicios de QualRep BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Países Bajos Teléfono: +31 20788 2630	
UDI BAR CODE (FPO)		
		

NATRACURE™ ADVANCED COOLING GEL CAP**REF 762-00NC1**

1 Piece | 11.8" x 7.5" x 2.0"

Designed to help

- Reduce hair loss*
- Soothe throbbing headache and pain during and after chemotherapy

Key features:

- New GEL/ICETM formula for long cold therapy treatments
- Insulated outer fabric helps retain the cold
- Rounded top for a more targeted treatment
- Fits heads 22.5" to 25.5"
- Suitable for freezer or refrigerator

Intended Use(s): For Cold Only: Provide cold therapy to specific body parts to support healing post-injury, surgery, chronic pain, and to help ease the side effects of chemotherapy.

Intended User(s): For Cold Only: For use by individuals with chronic pain, arthritis, recent or acute injuries, post-operative pain or individuals undergoing chemotherapy treatment.

Directions: Place the cap in the refrigerator or freezer, either wrapped in a paper towel or placed inside the accompanying plastic cover. Chill for a minimum of 2 hours. Place cap on the head and use the top hook-and-loop to adjust the fit, and bottom hook-and-loop for ponytails to leave the cap. Keep on for 20 minutes at a time.

Care: Hand wash the outer and inner fabric with soap and water. Air or blot out until completely dry. Use soaps that have less scent and other possible additives to minimize any possible reactions. Dispose of in accordance with local regulations.

Contraindication(s): Individuals using pacemakers or insulin pumps should NEVER use these products. Do not use during pregnancy. Do not apply cold to area of lost sensation. Do not use where there is poor circulation.

Warning(s):

1. FOR COLD USE ONLY!
2. Do not place product on an open wound.
3. If product causes skin irritation, discomfort or poor circulation occurs, discontinue use and notify your physician.
4. If you have diabetes, neuropathy, or poor circulation, consult a medical professional before using.
5. DO NOT STEP ON PACKS! Excessive weight may cause gel pack(s) to leak or rupture.
6. USE OVER CLOTHING OR PLACE FABRIC INTERFACE BETWEEN THE COLD PACK AND THE SKIN, PRIOR TO USING COLD THERAPY. POSSIBLE INTERFACES INCLUDE: A light towel, cloth, sheet/pillowcase (cut to size), 6 layers of paper towel, or Stockinette. Either wrap the interface around the hot/cold pack prior to attaching to the support (if included) or hold the interface on the skin while placing the product on the body.

Caution(s):

1. Consult your physician before using this product.
2. Only use this product for 20 minutes at a time on the affected area.
3. If at any time soreness, rash, or an increase in pain occurs, discontinue use and consult your physician.
4. Only use this product for its intended purposes as described in the instructions for use.
5. Do not use this product while using transdermal drug delivery skin patches, skin creams, balms, liniments or lotions.
6. If removable pack has been torn or has been compromised (and is leaking) discard immediately.
7. Always use a fabric interface.
8. Keep out of reach of children and pets.

Materials: PE/Nylon Loop Velvet, PTFE-coated nylon, Proprietary Gel Formula

UMDNS: 25806

Made in China

©2024 PolyGel®, LLC

 PolyGel, LLC

1 Danforth Drive, Easton, PA 18045 USA

NATRACURE™ is a brand of PolyGel, LLC

NatraCure.com

QualRep Services BV

Utrechtseweg 310 – B42, 6812 AR, Arnhem – the Netherlands

Phone: +31 20 788 2630

globalreg@qservegroup.com

* Studies that cite cold therapy for neuropathy pain and hair loss

<https://pubmed.ncbi.nlm.nih.gov/19171552/><https://pubmed.ncbi.nlm.nih.gov/32036471/><https://www.breastcancer.org/research-news/cold-therapy-helps-hair-skin-nail-effects><https://jamanetwork.com/journals/jamadermatology/article-abstract/2711245>

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Warning(s):

37. FOR COLD USE ONLY!
38. Do not place product on an open wound.
39. If product causes skin irritation, discomfort or poor circulation occurs, discontinue use and notify your physician.
40. If you have diabetes, neuropathy, or poor circulation, consult a medical professional before using.
41. DO NOT STEP ON PACKS! Excessive weight may cause gel pack(s) to leak or rupture.
42. USE OVER CLOTHING OR PLACE FABRIC INTERFACE BETWEEN THE COLD PACK AND THE SKIN, PRIOR TO USING COLD THERAPY. POSSIBLE INTERFACES INCLUDE: A light towel, cloth, sheet/pillowcase (cut to size), 6 layers of paper towel, or Stockinette. Either wrap the interface around the hot/cold pack prior to attaching to the support (if included) or hold the interface on the skin while placing the product on the body.

Caution(s):

49. Consult your physician before using this product.
50. Only use this product for 20 minutes at a time on the affected area.
51. If at any time soreness, rash, or an increase in pain occurs, discontinue use and consult your physician.
52. Only use this product for its intended purposes as described in the instructions for use.
53. Do not use this product while using transdermal drug delivery skin patches, skin creams, balms, liniments or lotions.
54. If removable pack has been torn or has been compromised (and is leaking) discard immediately.
55. Always use a fabric interface.
56. Keep out of reach of children and pets.

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NATRACURE™ är ett varumärke från PolyGel, LLC