

Kasutusjuhend küünarliigese tugi- mitte EBM

kubitaalkanali sündroom

night splint

day splint

immobilisation arm IV perfusion

Characteristics

universeal: left arm = right arm

left elbow = right elbow

(ambidextruous)

Sizes : universal

Black colour

Internal removable and moldable inner
aluminium splint

Body part: splint for arm-elbow

Art.nr. EBM-CTS-ET-1

Site web:

<https://www.belgomedical.com/store-20/item/ulajasemed/kuunarnukk/kuunarliigese-ortoos-kubitaaltunneli-sundroom/>

Guidelines (example-näide for right arm-elbow/parem käsi)

Place the splint in front of you on the
table.



logo EBM at the left side



slide the right arm into the inner sleeve



inner elbow well centered in the middle



Close upper velcro (shoulder side)



Close distal velcro (wrist side)



The flexible internal splint (you can
bend it if necessary) is placed beside
the arm, **not** under the elbow.



The arm is progressively slowed in the possibility of
bending (flexing) the arm to permit the healing of the
cubital tunnel nerve (about 60° flexion +/-)
It is 'possible' to immobilize the arm straight or in a
determined angle in placing the alu-splint in the elbow fold.
(not on the side), but this is normally not 'indicated' for
Cubital tunnel syndrome.

Volitatud edasimüüjale

Evidence Based Materials Bvba

Guido Gezellelaan 36/2

3550 Heusden-Zolder

Belgia

Tel 0032 11.53.77.47

<https://www.belgomedical.com/store-20/>



Küünarliigese tugi-mitte immobilisaato EBM



- cubitak tunnel syndrome elbow splint arm