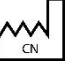

| | |
|---|---|
| NATRACURE™ COLD THERAPY SOCKS 1 Pair Large / Extra Large | REF 707 CAT |
| NATRACURE™ KOLDTERAPI SOKKER 1 par Stor / ekstra stor | CHAUSSETTES DE THERAPIE PAR LE FROID NATRACURE™ 1 Paire Grand / Très Grand |
| NATRACURE™ SOCKEN FÜR KÄLTETHERAPIE 1 Stück Large / Groß / Extragroß | NATRACURE™ CALZE PER TERAPIA DEL FREDDO 1 Paio Grande / Extra Large |
| NATRACURE™ KULDETERAPISOKKER 1 par Stor / Ekstra stor | [NATRACURE™ MEIAS PARA TERAPIA DE FRIO 1 Par Grande / Extra Grande |
| CALCETINES DE TERAPIA EN FRÍO NATRACURE™ 1 par Grande/Extra grande | [NATRACURE™ STRUMPOR FÖR KYLTERAPI 1 par Stor / Extra Stor |

| | | |
|--|--|---|
|  QualRep Services BV Utrechtseweg 310 – B42, 6812 AR, Arnhem – the Netherlands Phone: +31 20 788 2630 globalreg@qservgroup.com |  PolyGel, LLC 1 Danforth Drive Easton, PA 18045 USA |  |
| QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Holland Telefon: +31 20788 2630 | QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Pays-Bas Téléphone: +31 20788 2630 |  YYYY-MM-DD |
| QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Paesi Bassi Telefono: +31 20788 2630 | QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Niederlande Telefon: +31 20 788 2630 |      |
| QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Holanda Telefone: +31 20 788 2630 | QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Nederland Telefon: +31 20788 2630 |  |
| QualRep Services BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Nederländerna Telefon: +31 20788 2630 | Servicios de QualRep BV Utrechtseweg 310 - B42, 6812 AR, Arnhem - Paises Bajos Teléfono: +31 20788 2630 | |

**NATRACURE™ COLD THERAPY SOCKS****REF 707 CAT****1 Pair | Large / Extra Large**

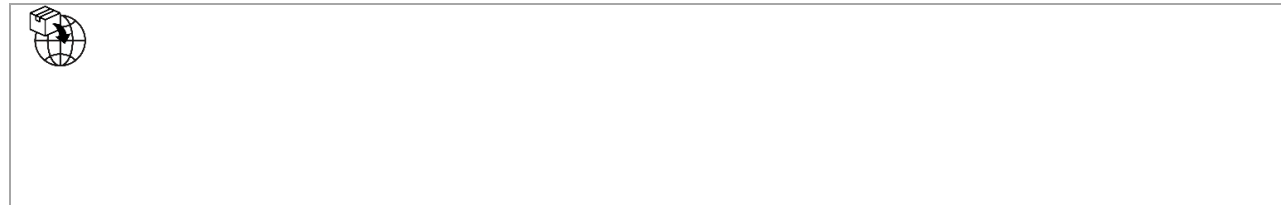To help relieve foot pain and discomfort. May help with the side effects of chemotherapy, arch pain, ball-of-foot pain, heel spurs, hot feet, sprains and strains, chronic conditions including arthritis, neuropathy, and plantar fasciitis.

Intended Use: Provide cold or heat therapy to specific body parts to support healing post-injury, surgery and/or chronic pain.

Intended User: For use by individuals with chronic pain, arthritis, recent or acute injuries, or post-operative pain.

DIRECTIONS-COLD: WARNING: ONLY CHILL GEL PACKS WHILE INSERTED INTO THE COLD THERAPY SOCKS.

Chill gel pack(s) for 2 hours in freezer or refrigerator. Do not use if you have decreased skin sensitivity or an open wound. If diabetic, consult a physician before using cold therapy. Do not exceed 20 minutes per use. Gel pack **MUST** reach room temperature if previously heated, before being chilled. Gel Packs can also be heated and offer warm relief for the heel, plantar fascia, forefoot and toes.

DIRECTIONS-HEAT: WARNING: DO NOT HEAT 'STAND ALONE' GEL PACKS! ONLY HEAT GEL PACKS WHILE INSERTED INTO THE COLD THERAPY SOCKS.

HEAT IN MICROWAVE FOR 10 SECONDS ON EACH SIDE.

DO NOT exceed 40 Seconds of TOTAL HEATING time per application of product. **DO NOT HEAT** more than 4 times in a 60 minute period. Gel packs **MUST** reach room temperature if previously chilled, before being heated.

*These products must be completely DRY and reach ROOM TEMPERATURE before switching from hot to cold therapy - or cold to hot therapy. This product has been designed to be used in an 800 WATT microwave. Temperatures of microwaves vary so adjust as necessary but **DO NOT** use with a microwave that exceeds 1400 WATTS.

CARE: Gel Pack - **DO NOT** machine wash or dry the Gel pack(s). Gel Packs should be hand washed only and must be completely dried before use. Store in a cool, dry place.

Discard if Gel Pack(s) are punctured or ruptured. Wrap - wipe down with damp (not wet) cloth, using mild soap and allow it to fully air dry before continuing use. Dispose of in accordance with local regulations.

Contraindication(s): Individuals using pacemakers or insulin pumps should **NEVER** use these products. Do not use during pregnancy. Do not apply heat to area of lost sensation. Do not use heated removable packs on parts of body where there is swelling, redness or immediately after injury. Cold therapy is recommended for these conditions. Do not use where there is poor circulation.

Warning(s):

1. **DO NOT OVERHEAT. OVERHEATING PRODUCT MAY RESULT IN SKIN BURNS.**
2. Do not use if you have decreased skin sensitivity or an open wound.
3. Due to differences in microwave ovens, heating temperatures may vary. Do not leave microwave unattended while heating.
4. If product causes skin irritation, discomfort or poor circulation occurs, discontinue use, and notify your physician.
5. If you have diabetes, neuropathy, or poor circulation, consult a medical professional before using.
6. **DO NOT STEP ON PACKS!** Excessive weight may cause pack(s) to leak or rupture.
7. **USE OVER CLOTHING OR PLACE FABRIC INTERFACE BETWEEN THE HOT/COLD PACK AND THE SKIN, PRIOR TO USING HOT OR COLD THERAPY. POSSIBLE INTERFACES INCLUDE:** A light towel, cloth, sheet/pillowcase (cut to size), 6 layers of paper towel, or Stockinette. Either wrap the interface around the hot/cold pack prior to attaching to the support (if included) or hold the interface on the skin while placing the product on the body.

Caution(s):

1. Consult your physician before using this product.
2. Only use this product for 20 minutes at a time on the affected area.
3. If at any time soreness, rash, or an increase in pain occurs, discontinue use and consult your physician.
4. Only use this product for its intended purposes as described in the instructions for use.
5. Do not use this product while using transdermal drug delivery skin patches, skin creams, balms, liniments or lotions.
6. If removable pack has been torn, over heated or has been compromised (and is leaking) discard immediately.
7. Do not use heat in direct contact with the eyes.
8. Always use a fabric interface.
9. Do not apply heat to the genitalia.
10. Keep out of reach of children and pets.

Material: COLD THERAPY SOCK: Microfiber Fabric. GEL PACK: Nylon laminated with PVC, Proprietary Gel Blend.

UMDNS: 25806

Made in China
©2023 PolyGel®, LLC
 PolyGel, LLC
1 Danforth Drive, Easton, PA 18045 USA
NATRACURE™ is a brand of PolyGel, LLC

QualRep Services BV
Utrechtseweg 310 – B42, 6812 AR, Arnhem – the Netherlands
Phone: +31 20 788 2630
globalreg@qservegroup.com

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NATRACURE™ är ett varumärke från PolyGel, LLC